FR – Description :

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EN – Description :

Transi-Mark® is a medical device consisting of 6 capsules in a blister pack each containing 10 radiopaque markers. These markers are to be taken orally and will not be digested or metabolized and will be evacuated naturally in the stool.

Composition :

The **Transi-Mark®** markers are composed of silicone impregnated with barium sulphate (43%) contained in vegetarian capsules composed of a Hypromellose membrane.

Indications :

Transi-Mark® is intended for use in adults and children with constipation or gastrointestinal disorders. It is a prescription-only test.

Instructions for use :

The capsules should be taken at the same time daily for 6 days. An abdominal x-ray should be taken on the seventh day at the time of taking the capsules. This means that the abdominal x-ray must be scheduled beforehand and the taking time adjusted according to the appointment. The capsules can be opened and the markers mixed with stewed fruit or yoghurt for patients who have difficulty swallowing the capsules.

Example with abdominal x-ray at 9:00 a.m.

Day	1	2	3	4	5	6	7
Time	09:00	09:00	09:00	09:00	09:00	09:00	09:00
Action	1 st Capsule	2 nd Capsule	3 rd Capsule	4 th Capsule	5 th Capsule	6 th Capsule	Abdomi nal X- ray

Warnings and Contraindications :

Transi-Mark® has no contraindications or drug interaction. It has no significant side effects. In the absence of data, this test is not recommended for use during pregnancy or lactation. Do not chew or bite. Do not take laxatives or other medications that affect colonic transit during the test. If you forget to take a test, contact your doctor.

Measuring Colonic Transit Time :

The technique is based on analysis of the abdominal X-ray of the migration of the markers into the colon. The physician can download a summary for the calculation and interpretation of the results on www.transimark.com or contact the customer service to receive the summary by email or mail.

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